



VenaCure EVLT

Endovenous Laser Treatment System



**NeverTouch
Gold-Tip Fiber**

NeverTouch Gold-Tip Fiber


Endovenous Laser Treatment

The NeverTouch Gold-Tip Fiber for Less Pain and Bruising

The NeverTouch gold-tip laser fiber establishes a standard for patient comfort, visibility and ease of use. The gold-tip fiber eliminates laser tip contact with the vein wall, which in turn minimizes perforations of the vein wall that typically result in pain and bruising as compared to traditional bare-tip fibers. The gold-tip also maximizes ultrasonic visibility, making it easier for physicians to use.

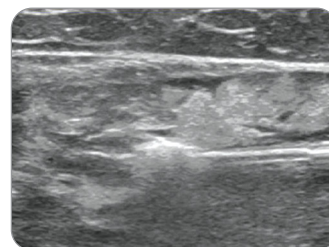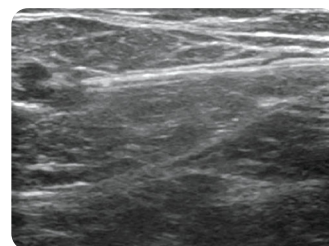


Echogenic

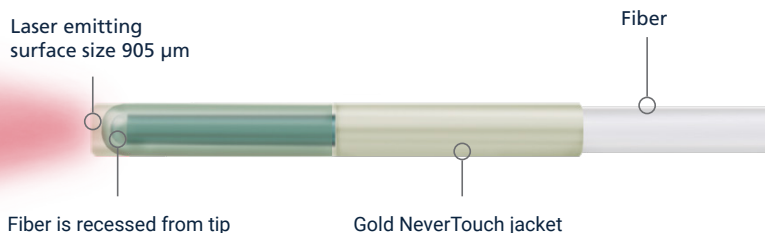

The Proprietary Science Behind NeverTouch Fibers¹

The innovation behind the NeverTouch fiber is a glass weld at the distal tip of a 600 μm fiber. The weld results in an effective fiber diameter of 905 μm and lowers the actual power density by 56 percent from that of a standard bare tip 600 μm fiber. The net effect is a homogeneous ablation with less focal charring of the vein wall than that which is seen with bare tip fibers. AngioDynamics' proprietary technology is different from competitors with standard covered-tip fibers that have the same power density as a 600 μm fiber.

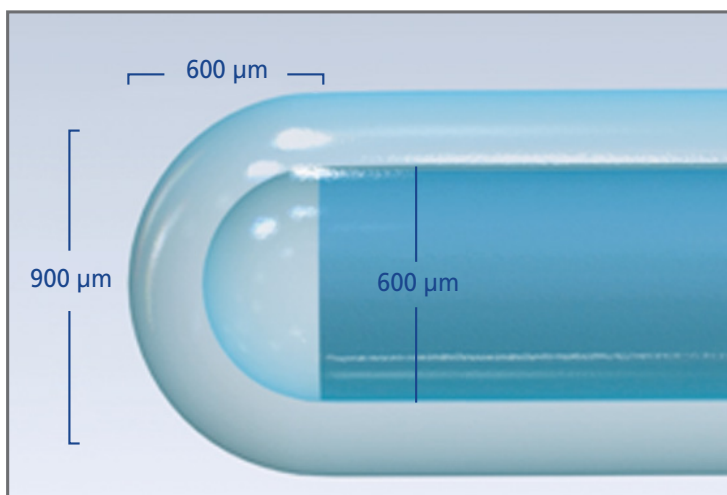

Procedure Kit Features and Benefits

Sheath-Lok system
Enables precise placement at the saphenofemoral junction by locking the fiber to the sheath, assuring the fiber stays a safe distance outside the sheath.




†Micro-access system in convenient "kangaroo pouch"
Select from a variety of micro-access kits, all packaged on the outside in a separate sterile pouch.



4F TRE-Sheath hydrophilic-coated introducer (translucent, radiopaque, echogenic)
Echogenic for easy positioning and hydrophilic coating to reduce procedure time.



Stainless steel braid
Improves pushability through tortuous anatomy and increases echogenicity.

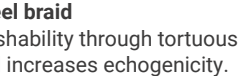




NeverTouch-FRS fiber
Fiber recognition system is compatible with all AngioDynamics' lasers.




Sliding sheath gauge
Aids in marking treatment length and holding sheath in place during tumescent anesthesia.

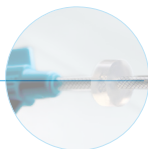

Available with a 21 or 19 gauge needle
New 19 gauge needle now available for physicians who prefer to insert the .035" wire directly through the needle.



90 cm TRE-Sheath Introducer
Reach new lengths with the 90 cm NeverTouch procedure kit—ideal for gaining access below the knee.

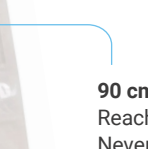

†Micro-Access System

AngioDynamics offers several micro-access systems to satisfy a variety of clinical needs. The chart below specifies the kit options and components that accompany the NeverTouch gold-tip fiber.

NeverTouch Fiber 0.018" System

The .018" system eliminates the need for a 5F micro-access sheath and eliminates the need to exchange wires and size up, resulting in fewer procedure steps, and a faster procedure time.

	21 G Needle	19 G Needle	0.018" Micro-access guidewire	Micro-introducer sheath with dilator	0.035" long-access double-ended guidewire	0.018" long-access guidewire	TRE-Sheath with dilator
Standard System	⊗		⊗	⊗	⊗		⊗
0.018" System	⊗					⊗	⊗
19 G †Micro-access System		⊗			⊗		⊗

AngioDynamics' VenaCure EVLT endovenous laser vein treatment offers patients a minimally-invasive choice for treating the source of their varicose veins and provides them with quicker recovery and a return to normal daily routines, as compared to surgical stripping. The VenaCure EVLT System includes a 1470 nm laser, 600 um or 400 um laser fiber procedure kits including accessories, marketing materials, support and more. For more information on these products, including our proprietary NeverTouch fiber technology, please visit www.VenaCure-EVLT.com.

NeverTouch Gold-Tip Fiber Procedure Kits

Description	UPN	Kit Length	Long-Access Guidewire	Needle
NeverTouch-FRS Kit	H787114030010	45 cm	0.035"	21 gauge
NeverTouch-FRS Kit	H787114030020	65 cm	0.035"	21 gauge
NeverTouch-FRS Kit	H787114040030	25 cm	0.018"	21 gauge
NeverTouch-FRS Kit	H787114030040	90 cm	0.035"	21 gauge
NeverTouch-FRS Kit	H787114030050	45 cm	0.035"	19 gauge
NeverTouch-FRS Kit	H787114030060	65 cm	0.035"	19 gauge
NeverTouch-FRS Kit	H787114030070	90 cm	0.035"	19 gauge
NeverTouch-FRS Kit	H787114030120	45 cm	0.018"	21 gauge
NeverTouch-FRS Kit	H787114030130	65 cm	0.018"	21 gauge

TRE-Sheath Introducer Set

Description	UPN	Kit Length
TRE-Sheath Introducer Set	H787114033015	45 cm
TRE-Sheath Introducer Set	H787114033025	65 cm



Scan the code to learn more about
NeverTouch Direct and procedure kit

AngioDynamics' VenaCure EVLT endovenous laser vein treatment offers patients a minimally-invasive choice for treating the source of their varicose veins and provides them with a potentially quicker recovery and a return to normal daily routines, as compared to surgical stripping.² The VenaCure EVLT System products include the VenaCure 1470 Pro Laser, a choice of procedure kits, procedure accessories, marketing materials and support. For more information on these products, including our proprietary NeverTouch fiber technology, please visit www.VenaCure-EVLT.com

Consult your local AngioDynamics representative for country specific availability.

1. 1. Based on Internal Test Report #525, Effect of Weld Length on NeverTouch Fiber's Beam Diameter. November 2011
2. 2. Brittenenden, J., et al. (2015, April). Clinical effectiveness and cost-effectiveness of foam sclerotherapy, endovenous laser ablation and surgery for varicose veins: Results from the Comparison of Laser, Surgery and foam Sclerotherapy (CLASS) randomised controlled trial. Health Technology Assessment, 19(27).

Important Risk Information

Indications for Use:

The VenaCure EVLT NeverTouch Procedure Kits, VenaCure EVLT NeverTouch Direct Procedure Kits and the VenaCure EVLT Tre' Sheath are indicated for endovascular coagulation of the Great Saphenous Vein (GSV) in patients with superficial vein reflux, for the treatment of varicose veins and varicosities associated with superficial reflux of the Great Saphenous Vein (GSV), and for the treatment of incompetence and reflux of superficial veins of the lower extremities.

This product should be used only with lasers cleared for use in the treatment of varicose veins, varicosities with superficial reflux of the GSV, and in the treatment of incompetent refluxing veins in the superficial venous system in the lower limbs.

Contraindications:

Contraindications include but are not limited to the following. Patients with thrombus in the vein segment to be treated. Patients with an aneurysmal section in the vein segment to be treated. Patients with peripheral arterial disease as determined by an Ankle-Brachial Index < 0.9. Patients with an inability to ambulate. Patients with deep vein thrombosis (DVT). Patients who are pregnant or breast feeding. Patients in general poor health. Other contraindications may be raised by the individual physician at the time of treatment.

Refer to Instructions for Use and/or User Manual provided with the product for complete Instructions, Warnings, Precautions, Potential Complications and Contraindications prior to use of the product.

CAUTION: Federal (USA) law restricts the sale of this device by or on the order of a physician.

USA > 14 Plaza Drive, Latham, NY 12110 > tel: 800-772-6446 or 518-798-1215 > fax: 518-798-1360
International > Haaksbergweg 75 (Margrietoren), 1101 BR, Amsterdam Z-O > The Netherlands
tel: +31 (0)20 753 2949 > fax: +31 (0)20 753 2939

www.angiodynamics.com | VenaCure-EVLT.com

AngioDynamics, the AngioDynamics logo, VenaCure, VenaCure EVLT, and the VenaCure 1470 PRO logo are trademarks and/or registered trademarks of AngioDynamics, Inc., an affiliate or a subsidiary. All other trademarks are property of their respective owners.
© 2025 AngioDynamics, Inc. GL/ID/BR/2960 Rev 02 07/2025

